



NPWT PUMP SETUP CHECKLIST

Demonstrate how to charge the pump	
Inform that the battery will last 7-18 hours depending on wound	
Confirm that the NPWT pump is operational	
Ensure that all supplies are within the expiration date and are undamaged	
Confirm that the number and sizes of the dressings are correct and correspond to the respective patients' therapy instructions as provided by their physician	
Review the alarms and how to resolve them	
Demonstrate how to change the canister and proper disposal of the canister	
If any liquid or food is spilled on pump, immediately wipe off	
Inform the patient that the canisters must be changed at least weekly and/or when canister is full. No canisters are to be reused	
Ensure Nirus NPWT Instructions for Use (pump user manual) is provided	